EN

SYMBOLS		
	Lot Number	Catalog Number
	Sterilised by Ethylene Oxide	Use Until Date
		Caution

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### INSTRUCTIONS FOR USE

#### Description:

The **Donor Trephine Punch** is a disposable punch and trephine system, for the trephination of the donor cornea button.

#### Intended Use:

The **Donor Trephine Punch** allows the surgeon to mark the four cardinal suture points and holds the donor corneal tissue in the most relaxed state in preparation for the cutting of the donor button. The **Trephine**, supplied as part of the system, is ultra sharp and makes a clean vertical cut through the corneal tissue.

#### CAUTION

- This device is intended for SINGLE USE ONLY. It is not recommended for repeat use.
- The punch is supplied sterile and ready for use. Do not use if the pouch has been opened or damaged.
- The **Trephine** is much sharper than a traditional trephine. Caution should be exercised at all times when handling and using the trephine.
- The **Trephine** is supplied with the blade in perfect condition. Should the device be accidentally dropped onto a hard surface, check the blade integrity under the microscope before use and discard the blade if damaged.

#### STERILISATION

- DO NOT RE-STERILISE. CorneaGen. assumes no liability for devices that have been re-sterilised by healthcare facilities.

#### SURGICAL TECHNIQUE

- Separate the two parts of the punch - the base and the trephine guide.
- Remove the cornea from the storage medium by grasping the rim of the surrounding sclera with toothed forceps and dry the anterior surface with a lint free PVA Sponge spear.
- Position the cornea in the block well with the endothelial surface up. Centre the cornea using the guide holes.
- Replace the trephine guide onto the base, being careful to maintain the centering of the cornea.
- Remove the trephine from the packaging and position in the guide, taking care not to damage the ultra sharp cutting edge.
- Confirm the position of the corneal tissue and if necessary use a micro hook instrument via the windows of the trephine guide, to manipulate and position the corneal tissue optimally.
- Lower the trephine blade gently onto the endothelial surface, and then press with a gentle but firm movement on the trephine to achieve the vertical cut through the corneal tissue. It is not necessary to rotate the trephine but if in doubt the trephine and trephine guide can be rotated together to ensure a thorough cut.
- Please note, The exceptionally super sharp trephine blade will cut through the corneal tissue without any sound or resistance. Once resistance is felt, the cut has been completed.**
- Once the cut has been achieved, remove the trephine and the trephine guide together in one piece. The corneal button, peripheral cornea and scleral ring will remain in the well of the base.
- In the unlikely event that the corneal button remains in the lumen of the blade, a balanced salt solution or a viscoelastic substance should be gently dripped into the centre of the blade to dislodge the button.

#### HAZARDS ASSOCIATED WITH THE RE-USE OF SINGLE USE ONLY DEVICES:

- Single use devices have not been validated for re-use. If you re-use a device you may be held **Legally Liable** for the safe performance.
- Cross-contamination and infection risks to patients . Including transmission of:
  - CJD & Variant CJD.
  - Prion Diseases.
  - Bacterial Endotoxins.
  - Hepatitis B & Hepatitis C.
  - Risks posed by HIV and AIDS
- Device failure through material fatigue or degradation caused by initial use and design:
  - Plastics:** Can be weakened, warped or become brittle.
  - Metals:** Can be damaged or subjected to rusting. Blade edges will be damaged or blunt. Needles may be bent from the original shape.
  - Other materials:** May degrade, becoming unacceptable when compared to original manufacturing criteria.
- Patient injury from device failure and/or chemical burns from residue of decontamination agents absorbed into the materials.

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© 2011 CorneaGen, Inc. All rights reserved.