

Preparation Instructions

If this is your first time using VisionGraft® or have difficulty locating the graft, please review instructions and short O.R. preparation video via QR code below. *If you have any questions, please contact us at 1-800-858-2020.*

Step 1



The outside of the peel pack is **NON-STERILE**. However, the inner vial and tissue are **STERILE**.

Step 2




A **STERILE** preparation basin will be needed for the **STERILE** field.

Step 3



CIRCULATOR, examine the peel pack. Do not use if there is evidence of damage or sterility has been compromised.

Step 4

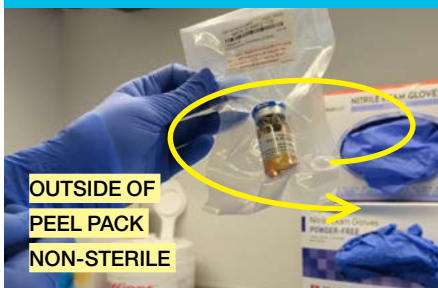

CIRCULATOR, gently swirl in front of a light to identify the tissue. It may adhere to cap or vial wall. If it cannot be found, try another peel pack and return the unopened one to CorneaGen for proper credit.

Step 5



CIRCULATOR, once tissue is visualized, open the **NON-STERILE** peel pack and let the **STERILE** vial drop onto the **STERILE** field.

Step 6



STERILE SCRUB, inspect the integrity of the **STERILE** vial and visualize the tissue. Gently swirl the vial until the tissue is identified. Tissue may adhere to the cap or inside wall.

Step 7



STERILE SCRUB, once tissue is visualized, remove the metal safety sealing band, the flip top and stopper.

Step 8



STERILE SCRUB, fast pour entire contents of the **STERILE** vial into **STERILE** basin on the **STERILE** field. Do not discard vial or vial cap yet.

Step 9



STERILE SCRUB, prior to discarding vial and vial cap, locate the tissue in the **STERILE** basin, once tissue is located, no rinsing is needed. Tissue is ready for the surgeon.



To access the *O.R. Preparation Video*, scan QR code or visit CorneaGen.com/VisionGraft